



BD GasPak™ EZ Gas Generating Systems

Unbeatable Convenience in a Complete Integrated System



Helping all people
live healthy lives

BD GasPak™ EZ Container Systems



BD GasPak® EZ Container Systems offer waterless, catalyst-free convenience for use in the innovative new GasPak EZ Incubation Containers or the original GasPak 100 or GasPak 150 jars. The GasPak EZ Gas Generating Container Sachets produce anaerobic, microaerophilic (reduced oxygen) or CO₂ enriched environments

EASY AS 1, 2, 3



1 Place inoculated BBL® plates in the GasPak EZ Petri dish rack and lower rack into GasPak EZ Incubation Container



2 Remove appropriate number of GasPak EZ Gas Generating Sachets and place in container (Use one BBL Anaerobe Sachet with Indicator when using the GasPak EZ Anaerobic System)



3 Latch GasPak EZ Incubation Container lid in place and incubate for culture

CONTAINER SYSTEM FEATURES:

- Two GasPak EZ Incubation Container Sizes (15 and 30 Petri dish capacity)
 - > Containers are non-breakable and chemical resistant for maximum safety
 - > Easy-close latches assure a secure, airtight seal
 - > Removable Petri dish racks ease workflow
 - > Containers are designed to hold standard test tube racks for greater convenience
- GasPak EZ Sachets are endorsed for use in both the new GasPak EZ Containers and the original GasPak 100 and GasPak 150 jars
- Sachets require no water or catalyst for activation
- Sachets are engineered to provide rapid gas generation for optimal bacterial growth
- Anaerobe sachet has indicator attached, eliminating a step to increase lab efficiency and save time

BD GasPak™ EZ Pouch Systems

BD GasPak EZ Pouch Systems offer the convenience of pouches integrated into complete kits with everything you need to generate a pouch based anaerobic, microaerophilic or CO₂ enriched environment

POUCH SYSTEM FEATURES:

- The only complete gas generation kit – pouches, sachets and indicators are all provided
- Large pouch capacity
 - > One to four Petri dish capacity for Anaerobe Pouch System
 - > Two Petri dish capacity for CO₂ and Campylobacter Pouch Systems
- One-step resealable pouch. No sealing bars needed!
- Custom multi-layer plastic pouch material preserves environmental integrity for maximum performance
- Sachets require no water or catalyst for activation
- Sachets are engineered to provide rapid gas generation
- Anaerobe sachets have indicator attached



EASY AS 1, 2, 3



1 Place inoculated plates inside resealable pouch.



2 Remove one GasPak EZ Gas Generating Sachet and place in pouch. (Add BBL™ Anaerobe Sachet with Indicator when using the GasPak EZ Anaerobic System.)



3 Seal pouch by pressing zipper together and incubate for culture.



BD GasPak™ EZ Gas Generating Systems

GasPak™ EZ Standard Incubation Container 15 Plates	260671
GasPak™ EZ Large Incubation Container 30 Plates	260672
GasPak™ EZ Anaerobe Container System Sachets with Indicator 20 Sachets Per Box	260001
GasPak™ EZ Anaerobe Container System Sachets 20 Sachets Per Box	260678
GasPak™ EZ CO ₂ Container System Sachets 20 Sachets Per Box	260679
GasPak™ EZ Campy Container System Sachets 20 Sachets Per Box	260680
GasPak™ EZ Anaerobe Pouch System 20 Sachets, Resealable Pouches w/indicator	260683
GasPak™ EZ CO ₂ Pouch System 20 Sachets and Resealable Pouches	260684
GasPak™ EZ Campy Pouch System 20 Sachets and Resealable Pouches	260685
GasPak™ EZ Standard Incubation Container Rack Holds 15-18 Petri Dishes	260673
GasPak™ EZ Large Incubation Container Rack Holds 30-33 Petri Dishes	260674
BBL™ GasPak™ Dry Anaerobic Indicator Strips Package of 100	271051
BBL™ GasPak™ 100 System	260626
BBL™ GasPak™ 150 Large Anaerobic System	260628


Complete Solutions for Anaerobic, Campylobacter and CO₂ Enriched Testing

- BD BBL™ Prepared Plated Media
 - CDC Anaerobe Agar
 - Brucella Agar
 - Campylobacter Agar w/ 5 Antimicrobics
- BD Vacutainer™ Anaerobic Specimen Collector
- BD Port-A-Cul™ Anaerobic Transport Systems
- BBL™ CultureSwab™ MaxV
- BBL™ CultureSwab™ Plus Collection and Transport Systems
- BBL™ QualiSwab™ Anaerobe Set
- BD GasPak™ EZ Gas Generating Systems
- BD Bio-Bag™ Disposable Environmental Chambers
- Indicators
 - Anaerobic and CO₂ indicator strips
- BBL™ Crystal™ AutoReader and BD BBL™ Crystal™ Anaerobe ID Kit



BD Diagnostics
7 Loveton Circle
Sparks, MD 21152-0999
800.638.8663
www.bd.com/ids

BD, BBL, Easy and all other trademarks are property of Becton, Dickinson and Company, ©2007 BD. 12547 - May 2007 - Printed in USA.

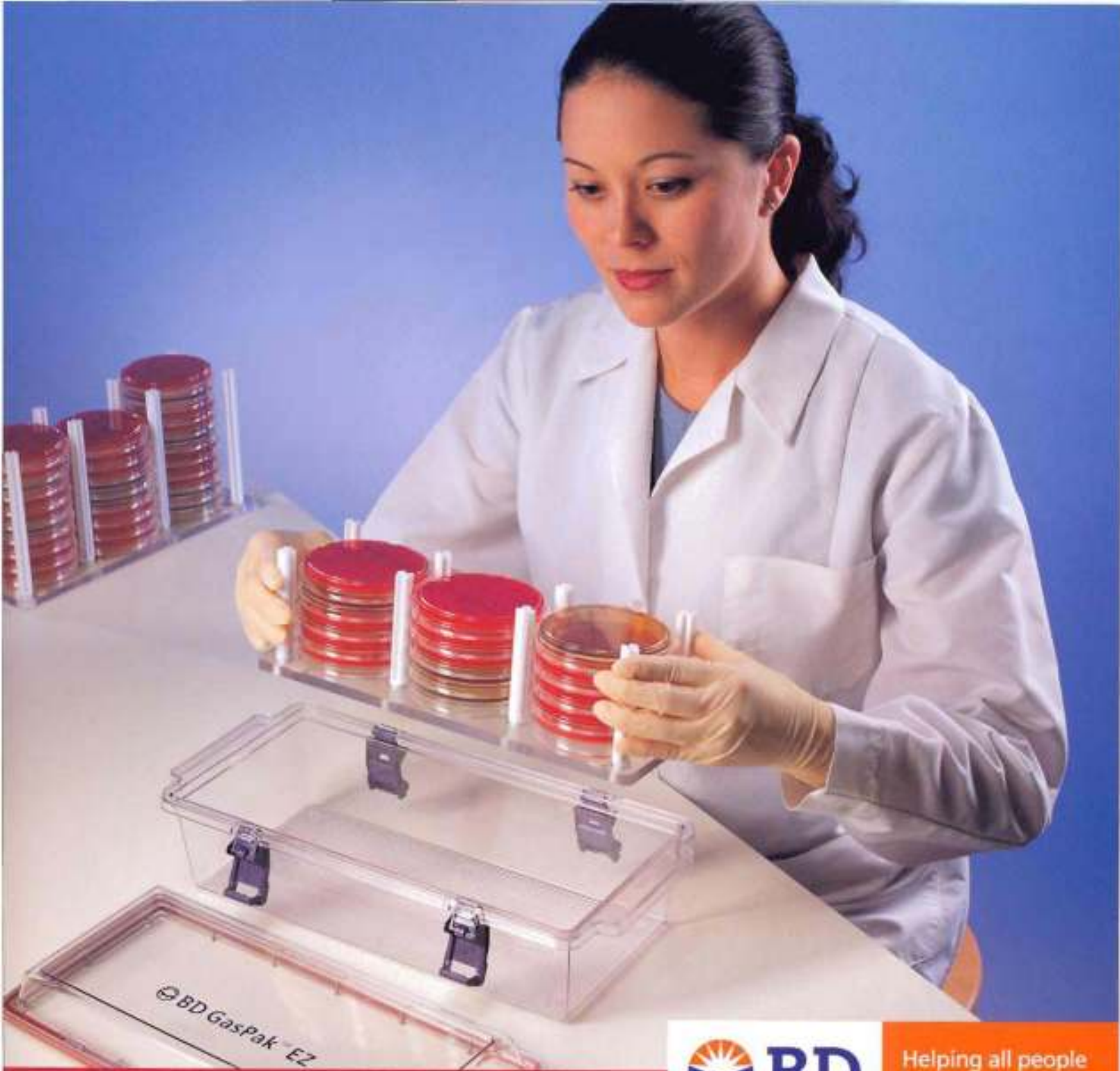

Helping all people
live healthy lives

BD GasPak™ EZ Gas Generating Systems

Unbeatable Convenience in a Complete Integrated System

BD GasPak™ EZ Gas Generating Systems

GasPak™ EZ Standard Incubation Container 15 Plates	260671
GasPak™ EZ Large Incubation Container 30 Plates	260672
GasPak™ EZ Anaerobe Container System Sachets 20 Sachets Per Box	260678
GasPak™ EZ CO ₂ Container System Sachets 20 Sachets Per Box	260679
GasPak™ EZ Campy Container System Sachets 20 Sachets Per Box	260680
GasPak™ EZ Anaerobe Pouch System 20 Sachets, Resealable Pouches and Dry Anaerobic Indicators	260683
GasPak™ EZ CO ₂ Pouch System 20 Sachets and Resealable Pouches	260684
GasPak™ EZ Campy Pouch System 20 Sachets and Resealable Pouches	260685
GasPak™ EZ Standard Incubation Container Rack Holds 15-18 Petri Dishes	260673
GasPak™ EZ Large Incubation Container Rack Holds 30-33 Petri Dishes	260674
BBL™ GasPak™ Dry Anaerobic Indicator Strips Package of 100	271051
BBL™ GasPak™ Disposable Anaerobic Indicator Package of 100	270504
BBL™ GasPak™ 100 System	260626
BBL™ GasPak™ 150 Large Anaerobic System	260628

Complete Solutions for Anaerobic, Campylobacter and CO₂ Enriched Testing

- BD BBL™ Prepared Plated Media
 - CDC Anaerobe Agar
 - Brucella Agar
 - Campylobacter Agar w/ 5 Antimicrobics
- BD Vacutainer® Anaerobic Specimen Collector
- BD Port-A-Cul™ Anaerobic Transport Systems
- BBL™ CultureSwab™ Plus Collection and Transport Systems
- BBL™ QualiSwab™ Anaerobe Set
- BD GasPak™ EZ Gas Generating Systems
- BBL™ GasPak™ Environmental Generating Systems
- BD Bio-Bag™ Disposable Environmental Chambers
- Indicators
 - Anaerobic and CO₂ indicator strips
- BBL™ Crystal™ AutoReader and BD BBL™ Crystal™ Anaerobe ID Kit



BD Diagnostic Systems
7 Loveton Circle
Sparks, MD 21152-0999
800.638.8663
www.bd.com/clinical

BD, BBL, Logo and all trademarks are the property of Becton, Dickinson and Company. All other marks are a trademark of Becton, Dickinson and Company, a subsidiary of Becton, Dickinson and Company. ©2009 Becton, Dickinson and Company. Printed in USA.

A commitment
to innovation
and performance

BD GasPak™ EZ Gas Generating Systems

In the late 1960s BD launched the first commercial system for environmental generation—BD GasPak jars and envelopes. This innovative line eliminated the need for vacuum pumps, gas tanks, nanometers and measure release valves required at that time to create an appropriate growth environment for microorganisms. Over the next 35 years BD continually introduced innovative products in response to our customers' evolving needs—including GasPak Plus envelopes, GasPak Pouch System, Bio-Bag™ Systems, anaerobic and CO₂ indicators, GasPak EZ Container and Pouch Systems and a complete line of accessories.

Today, in keeping with our history of innovation, BD is pleased to present the GasPak EZ Gas Generating System. The GasPak EZ System represents the first and only complete, integrated system for rapid, waterless and catalyst-free gas generation.



B
G

Gas
15

Co

Gas

20

Gas

20

Gas

20

Gas

20

Gas

20

Gas

20

Gas

20

Gas

20

Gas

20

Gas

20

Gas

20

Gas

20